# Buccal Swab Sample Collection Instructions

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The **Buccal Swab Sample Collection** should contain these items:

Three swabs

Specimen bag  
with label

Write your **name, date of birth, and collection date in MM/DD/YYYY format** on the specimen bag label in the designated area. Note that **you should not eat, smoke, or drink caffeinated beverages for 30 minutes prior to collection of the buccal sample.**

## Collect your sample.

### Rinse your mouth

Rinse your mouth with cold water, then wash your hands

### Open a swab and collect sample

Remove a swab from its wrapper. Firmly rub the swab on the inside of your cheek for 20 seconds in a circular motion. Dry the swab by shaking it in the air for a few minutes, and then place it into the specimen bag.

### Repeat for other swabs using the inside of the opposite cheek

Repeat the same process for the next two swabs, but use the inside of the opposite cheek.

### Package sample

Place the samples inside the biohazard bag provided, and close the bag tightly. Make sure you have written your first and last name, date of birth, and sample collection date on the label on the biohazard bag. Keep the sample out of extreme temperatures (above 100F or below 32F) before you ship.

## Ship your sample.

Once you have completed collection, simply place the biohazard bag with your sample back into the kit box. Then place the box back into the prepaid envelope and ship.

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